

Patient Settings

Setting	Description	Range
Weight	This indicates when you would like a patient to enter his or her current weight. Select "Pre" if you would like the patient to enter his or her weight during pre treatment and select "None" if you do not want the patient to enter his or her weight.	<ul style="list-style-type: none">• None• Pre
Weight Displayed in Tenths	Specifies if you want tenths of a weight unit to be displayed. Appears only if Pre is selected from the Weight dropdown list.	<ul style="list-style-type: none">• No• Yes
Night Therapy	This indicates that treatment will occur during the night.	Yes
Blood Pressure	This indicates when you would like a patient to enter his or her current blood pressure. Select "Pre" if you would like the patient to enter his or her blood pressure during pre treatment and select "None" if you do not want the patient to enter his or her blood pressure.	<ul style="list-style-type: none">• None• Pre
Manual Exchange Drain Volume	This indicates if you would like the patient to enter the volume of solution that the patient drained during manual exchange. This volume must be measured and entered in mL.	<ul style="list-style-type: none">• None• Pre
Manual Exchange Time	This indicates if you would like the patient to enter the time of day that the manual exchange was performed.	<ul style="list-style-type: none">• None• Pre
Manual Exchange Fill Volume	This indicates if you would like the patient to enter the Fill volume used for manual exchanges.	<ul style="list-style-type: none">• None• Pre
Manual Exchange Concentration	<p>This indicates the solution concentration type used for manual exchanges. When this setting is set to Pre on the device, it will initiate a patient survey question at the start of therapy to ask the patient to choose the solution concentration or solution type used during therapy. The selection option that the patient will have available is based on the following settings:</p> <ul style="list-style-type: none">• Solution Type (Dianeal and/or Physioneal)• Extraneal• Nutrineal	<ul style="list-style-type: none">• None• Pre

Patient Settings

Setting	Description	Range
Night Concentration	<p>This indicates the solution concentration type used during the night exchanges performed on the cycler. When this setting is set to Pre on the device, it will initiate a patient survey question at the start of therapy to ask the patient to choose the solution concentration or solution type used during therapy. The selection option that the patient will have available is based on the following settings:</p> <ul style="list-style-type: none">• Solution Type (Dianeal and/or Physioneal)• Extraneal• Nutrineal	<ul style="list-style-type: none">•None•Pre
Last Fill Concentration	<p>The Last Fill Concentration is for the strength or type of dialysis solution used for the last Fill. The Last Fill Concentration may be different from other supply bag concentrations used for night therapy. The strength or type of dialysis solution is prescribed by your nephrologist. When this setting is set to Pre on the device, it will initiate a patient survey question at the start of therapy to ask the patient to choose the solution concentration or solution type used during therapy. The selection option that the patient will have available is based on the following settings:</p> <ul style="list-style-type: none">• Solution Type (Dianeal and/or Physioneal)• Extraneal• Nutrineal	<ul style="list-style-type: none">•None•Pre
Solution Type (Dianeal and/or Physioneal)	<p>This allows the clinician to select if glucose or dextrose solutions will be used in the patient's treatment.</p>	<ul style="list-style-type: none">•Dextrose (1.5%, 2.5%, 4.25%)•Glucose (1.36%, 2.27%, 3.86%)
Extraneal	<p>This allows the clinician to determine if an Extraneal solution is used during a patient's therapy.</p>	<ul style="list-style-type: none">•No•Yes
Nutrineal	<p>This allows the clinician to determine if a Nutrineal solution is used during a patient's therapy.</p>	<ul style="list-style-type: none">•No•Yes