

## System Settings

Setting	Description	Range
Flush Before Fill	Determines whether the patient's cyler flushes solution in the cyler's heater line, supply line(s), and last Fill line (if used).	<ul style="list-style-type: none"><li>• Yes</li><li>• No</li></ul>
Fluid Temperature (Celsius)	Use this drop-down list to select the temperature of the heater bag fluid.	<ul style="list-style-type: none"><li>• 35</li><li>• 36</li><li>• 37</li></ul>
Two Chamber Bag	Use this drop-down list to tell the system if a two chamber bag is used.	<ul style="list-style-type: none"><li>• No</li><li>• Yes</li></ul>
Reset Patient Weight	Use this drop-down list to tell the system if the patient's weight should be reset to the default value at the beginning of every therapy.	<ul style="list-style-type: none"><li>• No</li><li>• Yes</li></ul>
Weight Units	Use this drop-down list to select the unit of measure for weight.	<ul style="list-style-type: none"><li>• kg</li><li>• lb</li></ul>
Adjust Loudness	Use this drop-down list to set the volume level.	<ul style="list-style-type: none"><li>• Low</li><li>• Medium</li><li>• High</li><li>• Maximum</li></ul>
Adjust Brightness	Use this drop-down list to set the brightness of the display screen.	<ul style="list-style-type: none"><li>• Low</li><li>• Medium</li><li>• High</li><li>• Maximum</li></ul>
Language	Use this drop-down list to select a language for text displayed on the screen.	<ul style="list-style-type: none"><li>• List of languages</li></ul>
Auto-Dimming of Display	Use this drop-down list to have the display automatically dim after treatment begins.	<ul style="list-style-type: none"><li>• Yes</li><li>• No</li></ul>